

LABORATORY VIBRATOR

Ray Foster

Model DV36

Operating and Maintenance Instructions



**Sales, Service, and Technical Assistance
Call Toll Free 800 / 654-4519**

Ray Foster

**Ray Foster Dental Equipment
5421 Commercial Drive
Huntington Beach, CA 92649
714 / 897-7795, Fax 714 / 898-6013
www.fosterdental.com**

IMPORTANT SAFEGUARDS

- Make sure unit is connected to a power source that is grounded.
- Never touch unit, cord, or plug with wet hands.
- Do not use if power cord or plug is damaged in any way.
- Do not pull on cord when unplugging unit.
- Keep unit away from heating instruments and other heat sources.
- Always unplug unit before cleaning. Do not immerse in water.
- **For laboratory use only.** This plaster vibrator is intended for use by trained professional laboratory personnel only. Use of the equipment in any way not intended by manufacturer compromises the safety of the equipment and voids the warranty.

OPERATING INSTRUCTIONS

1. Plug unit into grounded 115 volt 50/60 hz power source.
2. Turn control knob clockwise for LOW - MEDIUM - HIGH speeds. Turn knob counterclockwise to turn off.
3. Turn unit off when not in use.

MAINTENANCE

There is no maintenance other than keeping the vibrator clean. It is a good idea to wipe the unit after each use before the plaster sets. The rubber top is easily removed for cleaning. It may be removed and taken to a sink for thorough cleaning.

You should not let the vibrator run if not actively mixing plaster. This practice will extend the life of the unit.

DO NOT IMMERSE UNIT IN WATER.

SPECIFICATIONS

Voltage: 115v 50/60hz

Dimensions: 6-1/2" diameter x 5" tall

Net weight: 8 lb, 10 oz (3.9 kg)

PARTS LIST

| Part No. | Description |
|----------|-----------------------------|
| H401 | Rubber top, 6-1/2" diameter |
| H402 | Metal table top |
| H414 | Rubber Bottom |
| H418 | Switch, 3-Speed |
| H405 | Copper Spring Bracket |
| H421 | Transformer |

Ray Foster

Ray Foster Dental Equipment
5421 Commercial Drive
Huntington Beach, CA 92649
714 / 897-7795, Fax 714 / 898-6013
www.fosterdental.com